



humotech.

Open Source Leg (OSL)

The Open Source Leg (OSL) is an untethered robotic prosthetic leg designed by the Open-Source Bionic Leg Project, based out of the University of Michigan. The fully open-source device is a low cost, high performance robotic knee and ankle system designed to facilitate comparison between control strategies and to streamline the field towards highly functional robotic prosthetic legs. Humotech has partnered with the OSL project to increase the accessibility and ease-of-use of the device in the research market by manufacturing the device and providing support services to researchers. The Humotech OSL is based on the University of Michigan's OSL v2.0.

sales@humotech.com | 412.301.5083

PROPERTY

SPECIFICATION

Active Degree of Freedom

Force / Torque Sensors (with Loadcell Kit purchase)
Angle Sensors

Range of Motion (Ankle)

Range of Motion (Knee)

Torque (Ankle) Rated / Peak (with Motors and Batteries Kit purchase)

Torque (Knee) Rated / Peak (with Motors and Batteries Kit purchase)

Peak Joint Speed (no load, with Motors and Batteries Kit purchase)

Battery Life (normal walking on level ground, with Motors and Batteries Kit purchase)

Device Weight (with Loadcell Kit and Motors and Batteries Kit purchase)

Device Dimensions

Usage Limits (with Motors and Batteries Kit purchase)

Battery Pack Voltage (with Motors and Batteries Kit purchase)

Ankle and knee joint control with onboard, geared electric motors
1x 6-axis load cell below the knee, also both motors give current feedback
1x hall effect encoder at each joint (14bit resolution)

Plantarflexion: 30 degrees
Dorsiflexion: 30 degrees

120 degrees (0 degrees is a straight leg)

29 Nm /145 Nm

29 Nm / 145 Nm

6.13 (rad/s) (at 33.3 VDC)

Approximately 2 hrs

Approximately 5.7 kg

Length ~26 cm
Width ~12 cm
Total Height ~58.6 cm

Maximum body weight at 2.5 mph walking speed 100 kg

33.3 VDC

OSL v2


Technical Specifications*

*This information is provided by the University of Michigan Neurobionics Lab based on their recommended components. Humotech supplies the system "as is" and makes no guarantees regarding performance, suitability, or specific outcomes.

humotech.

OSL v2

Assembly Schematic*

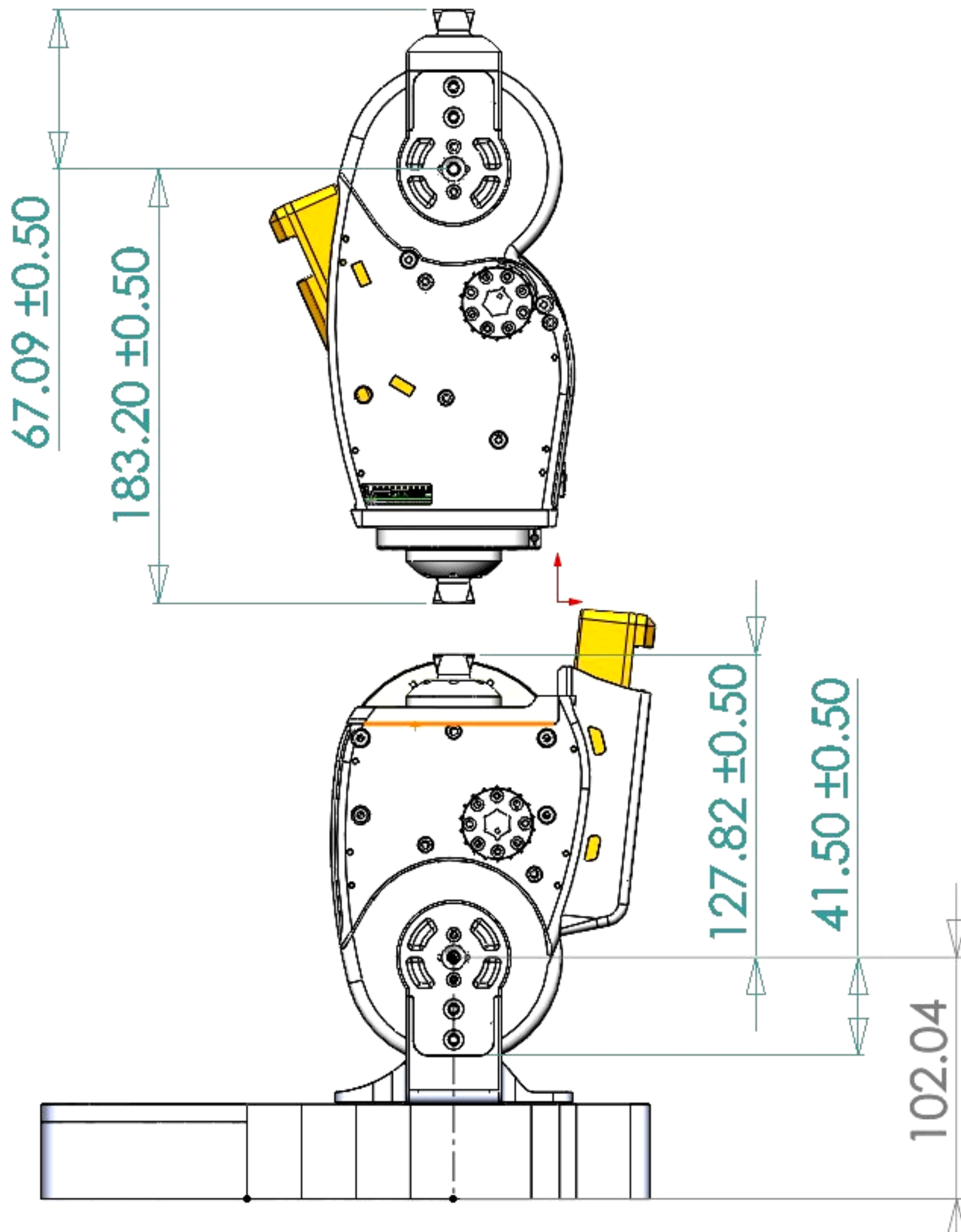

*This information is provided by the University of Michigan Neurobionics Lab using recommended components. Humotech supplies the system 'as is' and makes no guarantees regarding performance, suitability, or specific outcomes.

sales@humotech.com | 412.301.5083

OSL v2

Menu

Menu Item	Description	Contains
Baseline OSL	This is the foundational platform of the OSL. All other options require this as a starting point. If you only need the mechanical components and plan to source your own motors, batteries, and force sensors, this is the right choice. All other menu items also assume this selection.	Mechanical components (housings, pulleys, bearings, belts, fasteners, flat feet, etc.). Electrical components (Raspberry Pi, an SD card, joint encoders, BA30 connectors, etc.).
Loadcell Kit	A 6-axis loadcell to collect data and/or use in your control loop. This is custom designed by the Neurobionics Lab and is assumed to be used in the publicly available software.	1 below knee 6-axis loadcell selected by Neurobionics Lab. 1 strain gauge amplifier designed by Neurobionics Lab. Mounting hardware and electrical connectors.
Motors and Batteries Kit	A motor and battery set for both the knee and ankle. These are the recommended components from the Neurobionics Lab, around which the rest of the OSL was designed. They are also assumed to be used in the publicly available software. If you don't want to modify parts or handle motor and battery selection, this is the option for you.	2 motors selected by Neurobionics Lab. 2 Batteries selected by Neurobionics Lab. 1 Smart Dock for Batteries. Mounting hardware and electrical connectors. Basic setup of the ActPack firmware as recommended by the Neurobionics Lab.
VariFlex Foot Kit	A prosthetic foot for use with your OSL! This is the recommended foot from the Neurobionics Lab. It allows you to fine-tune the performance of the OSL through shoe size and stiffness selection.	1 foot with your choice of size and stiffness category. 1 left and 1 right foot shell, with your choice of color and mounting hardware.
Carrying Case	1 hard shell carrying case large enough for your OSL.	1 hard shell carrying case with custom foam.
Extra Battery	An extra battery for your Motors and Batteries Kit so you don't have to wait for them to charge.	1 Battery selected by the Neurobionics Lab.
Extra Smart Dock	An extra Smart Dock for your Motors and Batteries Kit so you can charge 2 batteries at once.	1 Smart Dock selected by the Neurobionics Lab.
Bent Knee Adapter	An adapter that allows use by individuals without an amputation. It is intended for testing your work before an end user wears it.	1 Bent Knee Adapter.
Lift Shoe	A shoe designed to pair with your Bent Knee Adapter and help maintain a more natural gait.	1 lift shoe in the size of your choice, see the Lift Shoe Options page for details.

All system components will be sourced by Humotech and can be warranted against manufacturing defects, except the Dephy motors, batteries, and battery chargers, as well as the Neurobionics Lab strain gauge amplifier, which are provided without warranty. All electromechanical components will be tested and verified to be fully functional in isolation; however, full system testing (e.g., walking) with Open Source Leg controllers is the customer's responsibility.

Lift Shoes are meant to be used in conjunction with a Bent Knee Adapter for controller testing purposes before attaching the OSL to an amputee. This allows researchers to walk on the device and solve any unexpected issues before collecting data. This table shows Humotech's recommendations for lift shoes based on the height of the individual who will be wearing the bent knee adapter and testing the OSL. This table IS NOT applicable to amputees.

OSL v2

Lift Shoe Options

Researcher Height Lift Shoe Recommendation For Use With A Bent Knee Adapter

<5' 6" (1.67m)	Tall lift shoe with custom length pylon strongly recommended
>5' 6" (1.67m) <5' 8" (1.72m)	Short lift shoe with 32mm pylon (included with Baseline OSL purchase)
>5' 8" (1.72m) <5' 10" (1.78m)	No lift shoe required! Just use the 32mm pylon (included with Baseline OSL purchase)
>5' 10" (1.78m)	No lift shoe required! Just use a custom length pylon

OSL v2

Service Options

Service	Description
Basic Technical Service Plan	Includes warranty and technical support.
Assembly Services	Assembly of purchased Open Source Leg components, as selected from the menu.
Training	Only available for the Assembled Open Source Leg.
Site Visit	Only available for the Assembled Open Source Leg.